

TRACOE Kids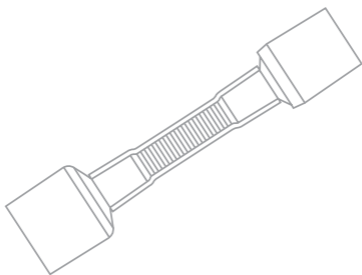

Dimensions, Maße, Dimensioni, Dimensioni,
Dimensiones, Medidas, Mål, Mitat
Dimensjon, Mått, Méretek, Méretek,
Διαστάσεις 尺寸, 치수,
داعبالا.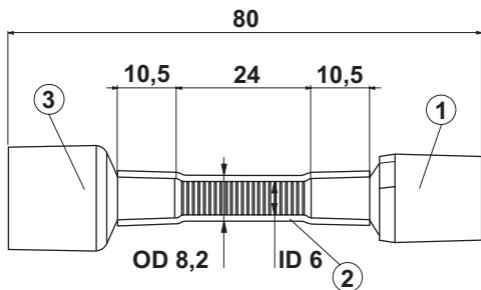

1. Connector 15 mm (male)
2. Reinforced tube
3. Connector 15 mm (female)

Contents

EN - ENGLISH.....	5
DE - DEUTSCH.....	9
NL - NEDERLANDS.....	13
FR - FRANÇAIS.....	17
IT - ITALIANO.....	21
ES - ESPAÑOL.....	24
PT - PORTUGUÊS.....	28
SV - SVENSKA.....	32
DA - DANSK.....	35
NO - NORSK.....	39
FI - SUOMI.....	42
CS - ČESKY.....	46
HU - MAGYAR.....	50
SL - SLOVENŠČINA.....	53
PL - POLSKI.....	56
RO - ROMÂNĂ.....	60
HR - HRVATSKI.....	64
BS - BOSANSKI JEZIK.....	68
SR - SRPSKI.....	72
EL - ΕΛΛΗΝΙΚΆ.....	76
TR - TÜRKÇE.....	80
KO - 한국어.....	83
ZHCN - 简体中文.....	86
AR - العربية.....	91

Instructions for Use for Accessories of Tracoe Mini Extension Piece REF 356

1. General Information

CAUTION: The following instructions are to be read carefully before using the product. The instructions only refer to the Tracoe Mini extension piece REF 356. All persons who are caring for children with tracheostomy tubes in hospitals or in the home care sector must be familiar with these instructions.

Sterilisation: The Tracoe Mini extension piece is sterilised with ethylene oxide. Sterility is only ensured if the packaging is intact and sealed on all sides.

Product for Single-Patient Use: The Tracoe Mini extension piece is a medical product, which is intended for only one patient. Multiple use on one and the same patient is permitted.

Useful Life: The manufacturer recommends that the Tracoe Mini extension piece should not be used for more than 29 days. Even pieces that show only minor damage must be replaced. Typically, the Tracoe Mini extension piece should be cleaned or changed daily.

Composition / Disposal: The Tracoe Mini extension piece consists of plastic materials that are internationally approved for medical applications. It contains neither latex nor DEHP. This product may only be disposed of in accordance with the valid national regulations governing waste materials.

Packaging Contents: Each package contains 1x Tracoe Mini extension piece, 1x disconnecting aid (wedge), 1x instructions for use and 1x product information card.

Product Information Card: On this product information card, there are two detachable labels, on which all relevant details of the product can be found. The product information card should be kept separately from the product. It will make reordering easier and is also essential in the event of a complaint when it must be enclosed

with the paperwork. The detachable labels can be attached, for example, to the patient's record.

2. General Description

The Tracoe Mini extension piece consists of a spiral-reinforced and DEHP-free PVC tube with a female 15-mm connector at one end and a male 15-mm connector at the other end.

3. Indications / Contraindications

3.1 Indications

The Tracoe Mini extension piece can be placed on all Tracoe Mini tubes. It is indicated for children with a very short neck in order to prevent the opening of the tube to be blocked. We recommend that the patient is monitored at regular intervals during the use of the Tracoe Extension Piece as should be done with all children with a tracheostomy tube.

CAUTION: The Tracoe Mini extension piece increases the dead space by approx. 3.5 to 4 ml (cm³). This roughly corresponds to the normal dead space of two to three-year old children.

3.2 Contraindications

The Tracoe Mini extension piece should not be used for children under the age of one or with a body weight of less than approx. 10 kg.

4. Instructions for Use

The female connector of the Tracoe Mini extension piece is pushed onto the male connector of the respective tube of the child. This prevents any possible closure of the 15-mm connector of the child's tube between chest and chin with children who have a short neck. The free male 15-mm connector allows for suction and ventilation if required.

CAUTION: If the child's breathing decompensates after the placement of the Tracoe Mini extension piece (this may be apparent from a blue colouring of the skin or from insufficient oxygen saturation levels), the Tracoe Mini extension piece must be removed immediately.

CAUTION: When using the Tracoe Mini extension piece, you need to make sure that the extension piece can easily be removed

from the tube. Both parts can be separated using the disconnecting aid (wedge) if required.

5. Care, Cleaning and Disinfection

The Tracoe Mini extension piece should be thoroughly cleaned at regular intervals. This can be done using commonly available domestic cleaning agents, such as mild detergents. For cleaning, we recommend the use of cleaning agents and accessories offered by Tracoe.

After it has been cleaned, the extension piece must be thoroughly rinsed with a sterile saline solution, with distilled water or with water of drinking water quality and then dried. For further details please see the instructions for use of Tracoe cleaning agents and accessories.

CAUTION: The Tracoe extension pieces must not be cleaned with aggressive domestic cleaning agents, cleaning agents for dentures or agents with a high alcohol content because these agents can impair the functioning of the products.

Do not heat-up, cook or steam-sterilise the extension pieces.

CAUTION: The Tracoe Mini extension piece must not be heated above 50 °C.

Disinfection: Since the upper airways are never free of germs even in healthy people, we recommend that you abstain from using disinfectants.

CAUTION: Do not use any disinfectants, which release chloride, or agents, which contain strong alkaline solutions or phenol derivatives.

Recommendation: We strongly recommend that you keep a clean and dry spare Tracoe Extension Piece available at all times.

6. Returns and Complaints

If you have a complaint about the device, please contact complaint.se@atosmedical.com.

If it is involved in a reportable incident, as defined in local medical device legislation, additionally contact the appropriate regulatory body in the country of use.

7. General Terms and Conditions

The sale, delivery and return of all Tracoe products shall be affected exclusively on the basis of the valid General Terms and Conditions (GTC), which are available either from Tracoe Medical GmbH or on our website at www.tracoe.com.

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© Coloplast A/S. All rights reserved.
Atos and the Atos Medical logo are trademarks of Coloplast A/S.